## Polyvet® Extra 100PE04B

### STANDARDS OF REFERENCE FOR PPE no 89/686/EEC

**Category I:** Minor risks. PPE of simple design for protection against minor risks which have no effects on the user safety.

**Category II:** Intermediary risks.

**Category III:** Major risks. PPE of complex design intended to protect against serious and irreversible health risks or mortal risks. Classified in different types:

**EN 14126:2003+AC:2004** Protection against infectious agents

**Type 3-B EN 14605+A1(2009)** – tight to chemical liquids and infectious agents

**Type 4-B EN 14605+A1(2009)** – tight to chemical spray and infectious agents

**Type 5-B EN ISO 13982-1+A1:2010** – tight to airborne solid particulates and infectious agents

**Type 6-B EN 13034+A1(2009)** – limited splash protection and infectious agents

### Coverall with elasticated hood:

- With self-adhesive zip-fastener
- Double adhesive flap at the fastening
- Elastic at the waist, ankles and wrists
- Thumb-loop
- Individual pouch

### Field of application:

- Protection against infectious agents
- Tight to chemical liquids
- Tight to airborne solid particles
- Tight to chemical spray
- Limited splash protection

A piece of protective clothing, single-use only, designed to protect the users against hazardous substance or potential contamination.

Protection against infectious agents according to EN 14126. The product protects against chemical liquids and airborne solid particles. Full body chemical and biohazard protection can be achieved only if this coverall is in combination with other PPE such as respiratory equipment, safety boots and gloves. Wrists and ankles need to be taped by adhesive (consult the manufacturer).

The garment does not contain any substance at levels known or suspected to affect the health or hygiene of the user in the foreseeable conditions of normal use.

### Conditions of use:

- Recommended to put on/off the PPE under the conduct of an experienced observer.

- Always zip up entirely
- Cover the boots and the gloves with the sleeves and the legs of the coverall.
- Adjust the sleeves and the legs by the elastic closures.
- It is recommended to cover by an adhesive tape all the openings at the face, sleeves and ankles.
- Put the first flap of the coverall on the second one by adjusting the double-sided adhesive gently, and then press the flap tightly. Move the double flap left and right to ensure that it is perfectly liquidtight. Attention: The protective paper of the double-sided adhesive has to be taken off before you fix the flap on the coverall.
- Type 5: Undress slowly the coverall by rolling it (inside out) or use the adequate substance for decontamination (attention to face, hands and feet).

### Limitations of use:

Exposure to certain very fine particles, intensive liquid sprays and splashes of hazardous substance may require coveralls of higher protection than that offered by Polyvet® Extra 100PE04B. Please ensure that you have chosen the garment suitable for your job and check if the chemicals and biohazard contamination you handle have been tested on the coverall (the use is limited to the biohazard tests and chemical product list enclosed).

Prolonged wearing of the coverall may cause thermic stress, please limit the duration of use.

For further information, please contact your supplier of the garment or Texsan Medical Bulgaria.

The user shall be the sole judge for the correct combination of Polyvet® Extra 100PE04B and ancillary equipment (gloves, boots, respiratory protective equipment etc). Highly important to follow the safety instructions for putting on/working with/putting off a PPE of the international and local health authorities.

Flammable: Stay away from flames or intensive heat.

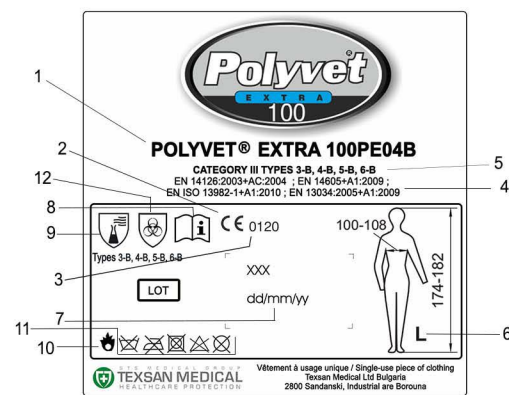
### Storage and Disposal:

Single-use piece of clothing. Discard after using.


- It should be stored in its original packaging, sheltered from direct sunlight and damp, at ambient temperature.
- Avoid storing this product at a temperature of -10° and +40°.
- Expiry date – up to 5 years after the date of production.
- Disposal: The polyethylene composing this garment and the product packaging present no danger for the environment and can be incinerated. Restrictions (according to the regulations in force) on the disposal depend solely in case of contamination during use.

The manufacturer shall not accept any responsibility for any improper use or disposal of garments.

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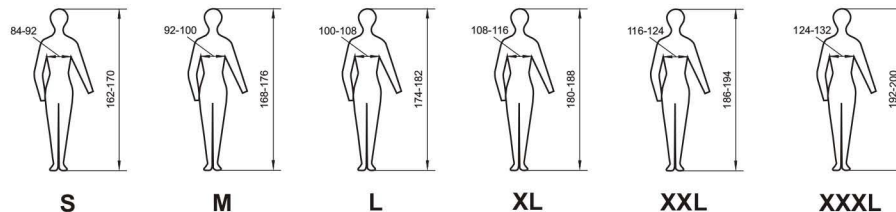Notify body for control and checking of PPE:

SGS United Kingdom Ltd. No O.N. 0120, Unit 202B, Worle Parkway,  
Weston-super-Mare, BS22 6WA UK

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1. Trademark
2. CE conformity marking signifies compliance with PPE of Category III according to the European legislation
3. EC Notify body number
4. Reference standard
5. Degree of protection
6. Size
7. Date of production
8. Pictogram «Information»- read the explanatory leaflet
9. Pictogram «protection against chemical products»
10. Pictogram «Flammable. Stay away from flames or intensive heat»
11. Pictogram «Do not wash, iron, machine dry, dry clean or bleach»
12. Pictogram «Protection against infectious agents»

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TF PE04B Annexe II  
edition 3.0 January 2015

## Polyvet® Extra 100PE04B

**EN COVERALL WITH HOOD FROM COPOLYMER OF POLYETHYLENE 100 µ, PROTECTION AGAINST INFECTIVE AGENTS**

**CATEGORY III TYPES 3-B, 4-B, 5-B & 6-B**

Size: S, M, L, XL, XXL, XXXL

Material and colour: Copolymer of polyethylene 100 µ, yellow

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